

OneTAB[®] Liquid

by ClinicalChoice

Frequently Asked Questions (FAQs)

Q: What is OneTAB?

A: OneTAB is a family of neutral pH, low foaming detergents used in the manual cleaning of surgical instruments and endoscopes. Enzymatic detergent formulas contain a powerful concentration of three different detergents (with surfactants) and three different enzymes that deliver maximum cleaning results.

Q: Why is OneTAB different?

A: OneTAB contains the power to clean. Its high concentration of protease, amylase and lipase enzymes and detergents clean porous and non-porous surfaces in harmony to lift and remove Blood, Fats, Protein, Synthetic Lipids, Carbohydrates, Biofilm, Polysaccharides, Simethicone, Oils & Cellulose-based adhesives.

Q: How do I use OneTAB Liquid?

A: OneTAB Liquid is dispensed the same as other liquid detergents, manually or mechanically. It can be dispensed at one-quarter to one-ounce per gallon.

Q: What departments can utilize OneTAB?

A: OneTAB can be used in any department that manually cleans endoscopes or surgical instruments, including ophthalmic instruments. Departments include GI/Endoscopy, SPD/Central Sterile, Operating Room, ENT, TEE, Urology and other outpatient departments.

Q: Is OneTAB a Green product?

A: Yes! OneTAB is environmentally and allergy friendly. It's manufactured with top-tier ingredients typically not utilized by other companies. It's 100% biodegradable and water soluble. It is designed without dyes, fragrances or caustic chemicals that can prohibit effectiveness and cause irritation.

Q: How do I convert to OneTAB?

A: The OneTAB family of products allows for complete conversion of detergents used for manual cleaning of endoscopes and instruments throughout your facility. This includes GI/Endoscopy, Operating Room, SPD and outpatient clinics. Clinical Choice will work with you to analyze current usage requirements and make recommendations on departments, facilities and IDN levels.

Q: Does OneTAB offer temperature storage and transport protection?

A: Yes! OneTAB Liquid maximum storage temperature is 150°F/65°C compared to 130°F/54°C or less on other liquid detergents.

Q: Do you provide product samples?

A: Yes! Please contact us at Clinical Choice, your local Clinical Choice representative or freetrials@clinicalchoice.com.

Q: Do you offer in-services?

A: Yes, please call Clinical Choice or your local representative to schedule and in-service.

Q: What type of on-line materials are available?

A: On-line materials include data sheets, FAQs, SDS' and IFUs.

Q: Is OneTAB safe for my staff?

A: OneTAB is designed for maximum cleaning. Although its environmentally and allergy friendly, personal protective equipment (PPE) should always be utilized. When used per IFU guidelines, OneTAB is equal to or safer than other cleaning products.