

## Instructions for Use (IFU)

### OneTAB SUPER ULTRASONIC™ 1–7 Gallons (OTSU-P)

#### Triple Enzymatic Detergent for Ultrasonic Cleaning of Surgical Instruments.

#### Directions for Use:

1. Open stick pack and evenly dispense OneTAB Super Ultrasonic Powder (OTSU-P) into an ultrasonic cleaner or sink containing between one to seven (1–7) gallons of water.
2. Soak and clean instruments for a minimum of two minutes to remove bioburden. Increase soak and cleaning time for instruments with visible bioburden.
3. Follow proper cleaning protocols set by your facility and society guidelines.
4. Remove instruments. Rinse thoroughly with tap, distilled, or sterile water.

The product's intended use is for manual cleaning of surgical instruments in an ultrasonic cleaner or sink. The product is not a high-level disinfectant and should only be used to clean devices prior to high-level disinfection or sterilization.

#### Applications:

- ▶ Ultrasonic and sink cleaning of surgical instruments.

#### Warnings and Precautions:

- ▶ The product should never be reused as this may lead to cross contamination.
- ▶ The product should only be used by qualified personnel with training in endoscope and surgical instrument reprocessing procedures. Personal Protective Equipment (PPE) should be worn when using this product.
- ▶ Original Equipment Manufacturer's guidelines regarding the acceptable care, cleaning and maintenance of each individual endoscope and surgical instrument should be reviewed prior to any cleaning procedures are performed.
- ▶ **CAUTION:** Do not consume or swallow the product. Avoid prolonged contact with skin. Keep away from children. Discard used solution. **DANGER:** Causes eye irritation. May cause skin and respiratory irritation, allergy or asthma symptoms or breathing difficulties if inhaled. Wear respiratory protection if ventilation is inadequate. **IF SWALLOWED:** Do not induce vomiting. Rinse mouth. **IF INHALED:** Breathe comfortably. **IF ON SKIN:** Wash with soap and water. If irritation occurs, get medical attention. **IF IN EYES:** Rinse carefully for several minutes. When comfortable, remove contact lenses and continue rinsing. **SEE SDS** for additional information.